

NEW!

BATRIK Tape-B-Gone™ a uniquely formulated and ready-to-use proprietary formula to easily remove adhesives from Medical Equipment and Instruments (All identification tapes).

Features

- Safe for use on metal, glass, plastic and rubber
- NOT flammable or combustible
- Neutral pH level of 7.0 - 8.0 with surfactants (trade secret)
- Easy to use "Spray Bottle"
- It is safe to use without any special equipment or ventilation
- Does not contain highly dangerous or caustic chemicals unlike many competitive brands
- We always recommend incorporating best hygiene and safety practices when handling any chemicals



Tape-B-Gone™

Item #	Description	Packaging
T-08RMV	8 oz (237 ml) Bottle	1 bottle
T-32RMV	8 oz (237 ml) Bottles	4 bottles per pack
T-SAMP-RMV	0.5 oz (15 ml) Bottle	1 bottle

Ask for a free sample today!

Transport Guidelines: Tape-B-Gone™ is not a dangerous goods and it is safe for Land Transport (DOT), Sea Transport (IMDG/IMO) and Air Transport (IATA).

Simple to use instructions: Sold in an easy spray bottle. Apply liberally to any label or adhesive. Allow to soak for 3-5 minutes. Lift corner and peel tape off slowly. If any glue residue remains, simply apply another application. Wait 2 minutes and wipe clean with a damp cloth and rinse off any remaining Tape-B-Gone™.