


# FIRSTBag

SEALED TRANSPORT SOLUTION

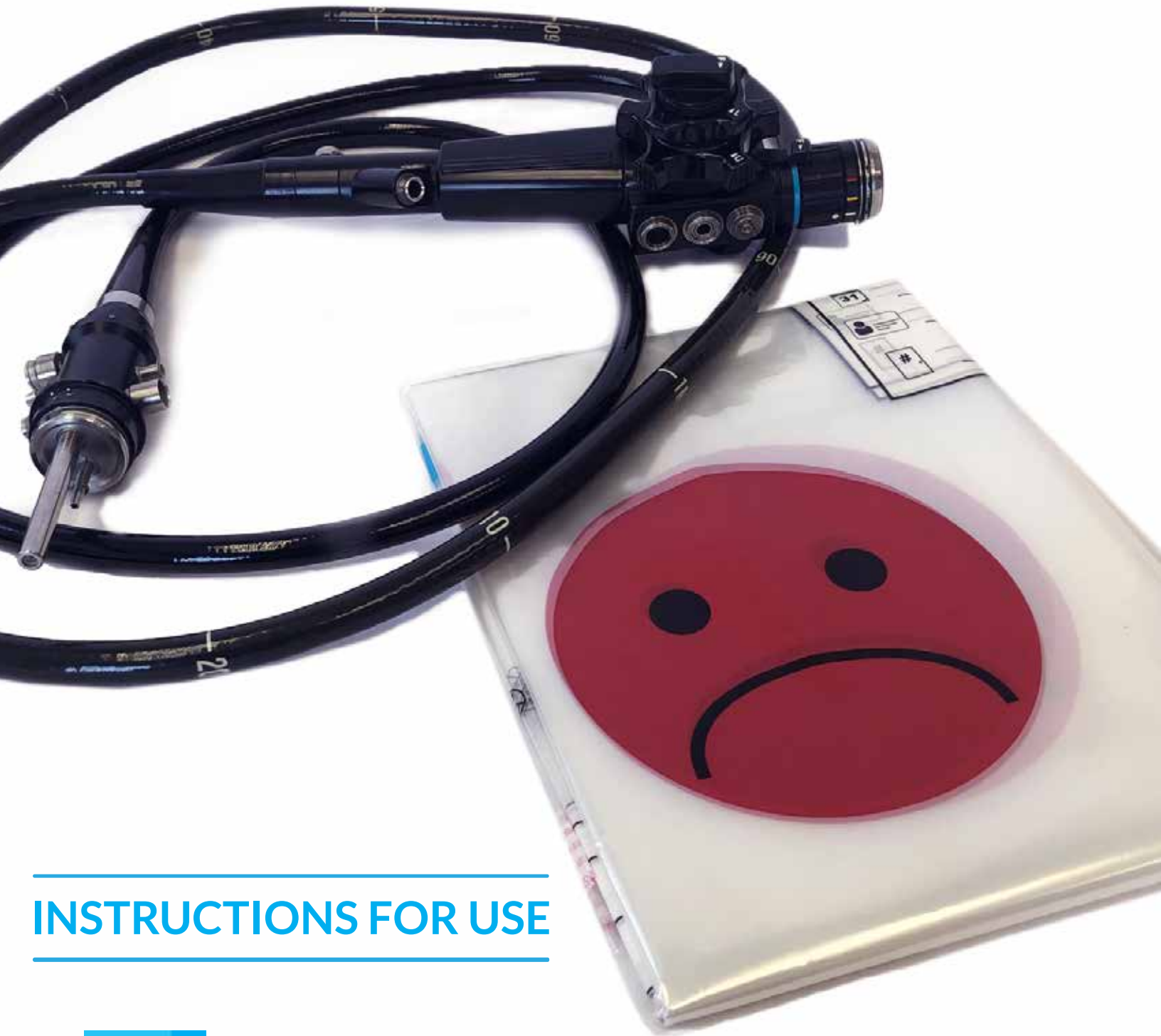

## INSTRUCTIONS FOR USE



**FIRST Bag SEALED TRANSPORT** bags are thoughtfully designed to eliminate touch-points during transport of endoscopes from point-of-care to the reprocessing area, and back, focused on safety of patient and medical staff. Two separate disposable clear plastic bags were designed for that purpose:

A bag with a red sad face and a biohazard symbol for soiled scopes and accessories that require reprocessing.


A bag for clean scopes and accessories ready for use, marked with a green happy face.

Using **FIRST Bag SEALED TRANSPORT** bags helps reduce user error, prevent cross-contamination and promote patient and medical staff safety.



## Made of recyclable LDPE material that:

- ☑ Can withstand high temperatures of up to 90 °C (194 °F)
- ☑ Is characterized by being quite flexible, highly resilient, and tough
- ☑ Excellent resistance to dilute and concentrated acids, alcohols, bases and esters
- ☑ Good resistance to aldehydes, ketones and vegetable-based oils\*

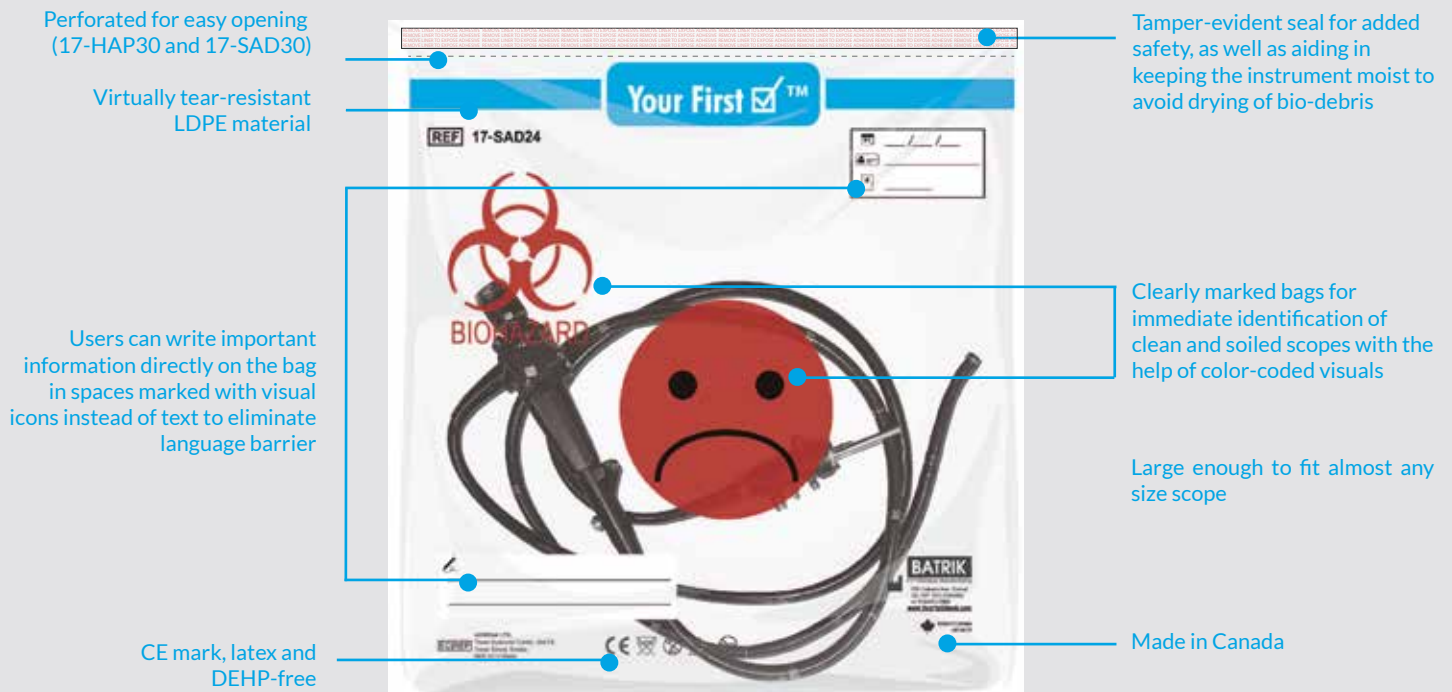

## Placing Medical Instruments in F1RST Bag SEALED TRANSPORT

1 Position an open F1RST Bag in the medical instrumentation bin



2

Using two hands, place the endoscope in the open bag



3

Peel off adhesive tape and fold the bag's flap to seal



## Placing Medical Instruments bin in F1RST Bag SEALED TRANSPORT

1

Place the medical instruments in the bin



2

Slide the bin in a F1RST Bag Sealed Transport




3

Peel off adhesive tape



4

Fold the bag's flap to seal

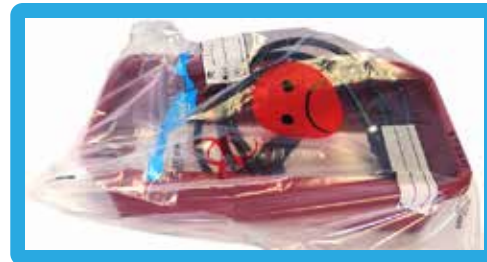

# RELATED PRODUCTS

Your First ™

## MEDICAL INSTRUMENTS PRE-CLEANING PRODUCTS

Our enzymatic bedside cleaning products are Your First Check in the instrument cleaning process. These include pre-clean foaming sprays, lint-free wipes, and sponges pre-treated with PrAmLi Power3 Enzymes™, offering a highly effective pre-clean or surface clean solution.



PrAmLi Power3 Enzymes™

## ENZYMATIC DETERGENTS

A unique blend of three powerful enzymes, along with a surfactant to provide the ultimate in cleaning efficacy.



SAVEFACE™

## FACESHIELDS

A combination of a flat viewing area to reduce distortion and glare, a self-shaping foam piece that molds to the forehead and a high quality 7.5 mil thickness PET offers the ultimate in comfort and safety for a faceshield that can be used for longer periods of time.



## BATRIK CLEANING BRUSHES

A wide variety of sizes and styles are available to meet your unique needs. With over a million brushes sold and ZERO defects, Batrik brushes provide superior quality for ease of use and peace of mind.

Packaging: Bag of 50



**FIRST**Bag  
SEALED TRANSPORT



### FIRST Bag SEALED TRANSPORT ORDERING INFORMATION

ITEM #	DESCRIPTION	SIZE INCHES / CM	PACKAGING
17-HAP24	For Clean Instruments	24" x 22" / 61 cm x 56 cm	200 Units / Case
17-SAD24	For Soiled Instruments	24" x 22" / 61 cm x 56 cm	200 Units / Case
17-HAP30	For Clean Instruments	30.5" x 26" / 77.0 cm x 66.0 cm	200 Units / Case
17-SAD30	For Soiled Instruments	30.5" x 26" / 77.0 cm x 66.0 cm	200 Units / Case

CONTACT US FOR MORE INFORMATION

[www.Your1stCheck.com](http://www.Your1stCheck.com)



info@surgmed.com



+1 514-631-7988  
+1-800-661-5432



850 Halpern Ave.  
Dorval, QC H9P 1G6