

Fight the "WHAT THE FOG?!" frustration with

WTFog™

by G.O.L.F.F.™

REUSABLE ANTI-FOG CLOTHS



WTFog™ by G.O.L.F.F.™ is a product by Batrik Medical Manufacturing Inc., a company with experience manufacturing surgical grade anti-fog products sold worldwide since 2005.



"What The Fog?!"

We've all experienced it.

Eyewear fogging up while wearing a mask is not only unpleasant and highly frustrating, but it can be dangerous, too.

INTRODUCING THE FOG FIX:



Our **WTFog™ reusable anti-fog cloths** prevent fogging on all types of eyewear. Ideal for use when wearing a mask or face shield, WTFog™ takes you from "What The Fog?!" to confidently ready for your next patient in a snap.

WHY YOU'LL LOVE IT:

- Eliminates fogging on all glass, polycarbonate, and plastic eyewear lenses
- Each reusable nano-infused cloth lasts up to 600 wipes or 6 months
- Microfiber cloths are infused with our proprietary heavy duty anti-fog treatment
- Quick and easy application - just wipe and wear
- Lint-free, scratch-free, no greasy smearing
- Long lasting protective coating provides effective protection against fog, humidity, sweat, and fast temperature changes.
- 100% Natural Titanium Dioxide (TiO₂) Nano Formula developed in Germany meets strict REACH standards
- Non-Sterile

PERFECT WHEN:

- You wear glasses/goggles and a mask/face shield
- Your profession regularly exposes you to fogging and you need crystal clear vision without the risk or hassle of fogging up



"To be able to complete a whole appointment without adjusting my eye protection makes maintaining COVID PPE protocols that much easier."

DR. MOLLY FRIEDMAN

HOLLAND BLOORVIEW KIDS REHABILITATION HOSPITAL
TORONTO, ON



DIRECTIONS FOR USE:

Open resealable pouch and remove the WTFog™ reusable anti-fog cloth. Wipe cloth on clean surface (glasses/goggle lens) being careful not to remove the protective coating delivered by the cloth. There is no wait time required before wearing your protected item. Enjoy your fog-free experience!

CLOTH STORAGE & CARE:

After use, keep cloth in the resealable pouch until next use, ensuring the seal is fully closed. If cloth becomes too dry, add up to 1 ml of water to pouch to reactivate cloth. For best results, store at ambient temperature/humidity. Dispose cloth after 600 uses or 6 months.

External surface use only.

Keep out of reach of children.

Do not wash.

PRODUCT SELECTION

ITEM #	DESCRIPTION	PACKAGING
WTFog-30	30 WTFog™ in a dispenser box measuring: 3 7/16" x 5 2/16" x 6 2/16" (8.7 cm x 13 cm x 15.4 cm)	30 WTFog™ / dispenser box
WTFog-100	100 WTFog™ - bulk packed	100 WTFog™ / box
WTFog-1200	1200 WTFog™ - bulk packed	1200 WTFog™ / case (12 boxes of 100 bulk packed)

1 WTFog™ Cloth per Resealable Pouch
WTFog™ Cloth: 5.5" x 5.5" (13.9 cm x 13.9 cm)
WTFog™ Resealable Pouch: 3.5" x 5" (8.9 cm x 12.7 cm)



2165 46th Ave., Lachine, Quebec H8T 2P1
Phone: +1 514-631-7988 | Toll Free: 800-661-5432
Email: info@surgmed.com | Website: www.batrik.com